



MESOMEGA

by Medixsysteme - France

FIRST WORLD DIGITAL PREMIUM
ELECTRONIC MESOGUN




Selecting the right mesogun is important for your clinic, let's give you some advices to enjoy the most of it and to make this tool your best assistant in mesotherapy treatment.....

Mesotherapy pratice from France since 1952.....



MESOMEGA to practice safe mesotherapy – CE Medical approved

Everything you need in one handy hard light suitcase, ideal to store and carry your mesomega wherever you like... Anonymously... no promotional name on the suitcase, making travelling more easy between your clinics.

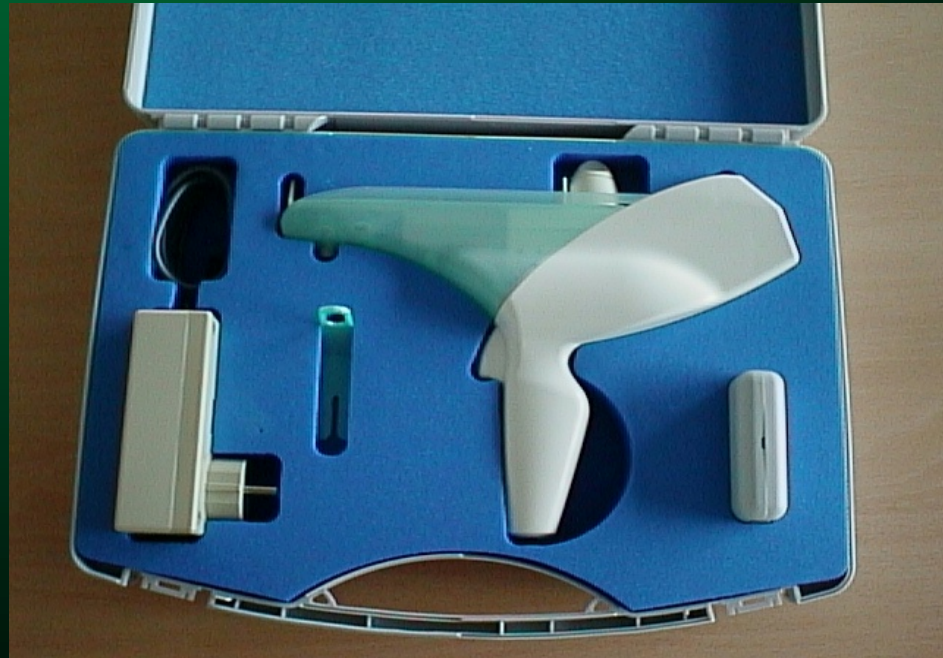



MESOMEGA Elements & unique registered Italian design

Re usable, Special Medical plastic for autoclave
Skin Stabilizer

Multi function cursors for dry and wet mesotherapy

Ergonomic lines for men and women doctors easy grip

Electronic and green LCD screen digital

European High Technology Mesogun


Unique Long Life Battery LITHIUM ION rechargeable over 3 years !!

Battery auto detection alarm

Charger

Able to cope with power variation from 100 volts to 240 volts... automatically adjusted...

Plug adapted to most country at **the order**

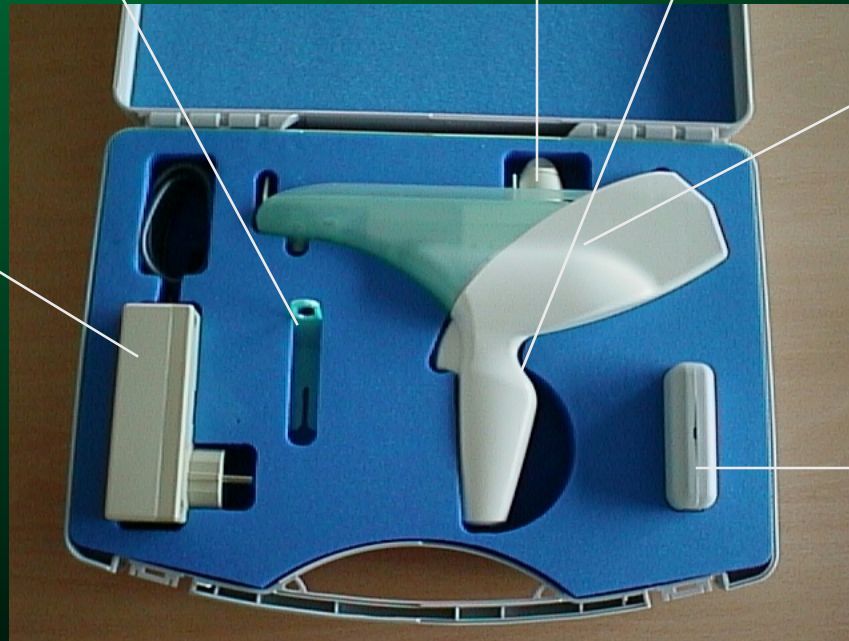



Major Functions of Mesomega

The control panel, will assist you to select and operate the various functions of injections, selection of syringes, but also to verify that the mesogun is on or off, and what is the battery level at all time

4 FUNCTIONS

Continuous

Dosimetric

Nappage

Mesoperfusion



LCD
Screen

Digital
Control
panel




MESOMEGA BASIC

Lithium Ion Battery, military tested for outdoor operation, given to be reloaded 1050 times (3 years) Easy access to battery, to be clip down with one finger only...

Perfect Hygiene in your clinic with its unique easy maintenance in just removing the front green part you can clean your Mesomega with simply clothe and alcohol.

One clic on unique smooth trigger... give perfect control and fast treatment.

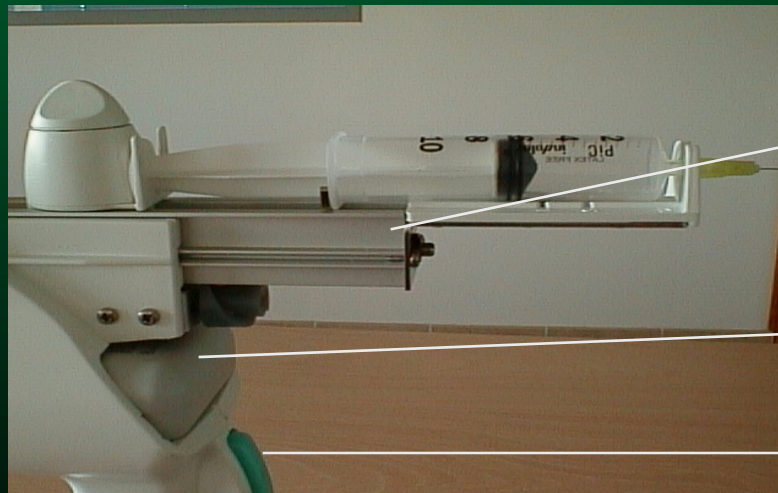

Easy access for cleaning so you get a perfect Hygiene

Battery

Smooth trigger



MESOMEGA for all meso

Using the key board allows you navigate in the FOUR programmes of Mesomega

- * Continuous where you can inject at your convenient speed with Mesogun automatically adjusting our pain control system to keep the comfort of the session to your patients. (3ml/mn to 8.4ml/mn)
- * Dosimetric able to determine at microliter path your ideal dose, strongly recommended for Mesobotox or Hyaluronic acid injection in delicate area
- * Nappage with perfect product delivery at each drop
- * Mesoperfusion specially design for Pain, Sport and Mass effect mesotherapy





MESOMEGA FLEXIBLE

The control panel is there to help you to select the mode of mesotherapy treatment you want to work with,

But also the Standard Syringe you currently work with... from your country...



Continuous

dosimetric

mesoperfusion

nappage

1 CC

2.5 CC

5 CC

10 CC

20 CC





Fix your syringe..and start professional treatment

- ++++ Easy international syringe fixing of current sizes
- +++ No Leakage with unique antidrop mechanic
- +++ No syringe jumping with "luerlock integrated" carrier
- ++ Deep injection available up to 10mm deep totally painless with movement pressure control



TOTAL CONTROL OF PRODUCT DELIVERY

SELECTING MESOMEGA

- ✔ 4 Functions electronic LCD screen gun
- ✔ CE Medical approved by European Notified body give safety
- ✔ Registered Ergonomic Design for professional mesotherapist
- ✔ Exact dose to give precision
- ✔ No need of Kit to give freedom and no additional cost
- ✔ No need of extra battery you have autonomy
- ✔ Training and references available in many countries in the world... long time mesogun manufacturing experience in origin country make Mesomega the right choice for your clinic.

